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⚠ These user instructions do not replace the instructions of use of the debritom*

DebriClip QuickCard - EN

Contraindications

- Malignant tumor type
- Open injury to vessels, unprotected exposed vessels
- Eyes, ears, nose
- Delicate vessels and structures, such as neurovascular bundles
- For complex or highly contaminated wounds
- Patients with HIV, hepatitis C or another contagious disease

Warnings with regard to the use of the device for the following conditions, since they are associated with higher risk:

- Patients with increased tendency to hemorrhage
- Contaminated wounds

If treatment is indicated for the above listed conditions (listed under warnings), this treatment should be done under medical guidance and supervision.


1. Description

DebriClip is an additional attachment to debritom* handpieces for reducing aerosols and collecting irrigation fluid with debris during mechanical wound cleansing. The detailed description can be found in the IFU of the debritom* system.

2. Preparation

Ready to use debritom* (according to instructions for use REF 9000.5503)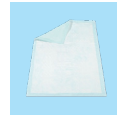

Recommendation: Prepare absorbent patient pad

3. Application

Basis: **ready to use debritom***

Remove DebriClip from packaging


Open sterile package

Snap the handpiece into the opening provided

The jet can be aligned by turning the handpiece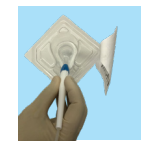

Remove the DebriClip from the packaging with the engaged handpiece

Apply the DebriClip compact to the wound and debride according to IFU 9000.5503.

⚠ Caution

Sterile disposable product: no re-use

Not to be used with the standard jet REF 2000.0003

Suitable for use with flat jet (wide and narrow).

for use with:
2000.0004
2000.0005

4. Additional functions

4.1 Suction

Remove closure

Select and connect suitable suction hose

Start the suction system (select and connect a suitable suction system with a free flow rate of 10l/min to 30l/min)

4.2 Mechanical cleaning

Clean the wound and wound edges mechanically with the sponge

5. Disposal

- Dispose the DebriClip with handpiece with or without suction hose
- Remove the DebriClip from handpiece and dispose of with or without suction hose.
- Dispose the DebriClip properly if there is a risk of contamination
- Obey in-house hygiene standard

medaxis swiss medical technology

medaxis AG

Bahnhofstrasse 9
CH-6340 Baar

T +41 62 823 88 00
F +41 62 823 88 01

www.medaxis.ch
info@medaxis.ch