





The Invia Motion Quick Card does not replace the Invia Motion instructions for use. For information concerning intended use, indications, contraindications, warnings and cautions please refer to the appropriate Invia Motion Clinician instructions for use and Invia Motion Patient instructions for use included with the Invia Motion pump or at [www.medela.com/healthcare](http://www.medela.com/healthcare).

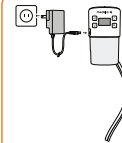

Fully charge battery for the first time before starting the therapy. Charging time can be up to 4 hours. Recharge battery throughout the length of the therapy. The expected battery time generally exceeds 10 hours.

## Overview

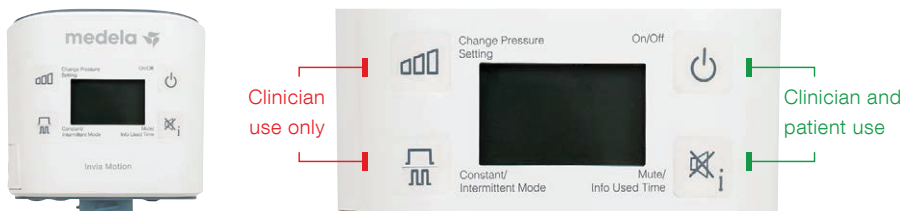

## Positioning and wearing Invia Motion Negative Pressure Wound Therapy (NPWT) pump



Upright



Laying with display side up



Upside down



Laying with display side down




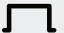






Shoulder strap




Belt strap

If placed in an inappropriate position or when canister is full, "Canister full" notification will appear and Invia Motion canister/tubing set must be changed to continue the therapy.


## Information on the display

Symbol	Description
	Pump is running/On-state in intermittent mode (Symbol is blinking)
	Constant mode
	Negative pressure setting in mmHg
	Battery charge level (symbol steady) Battery charging (bars moving)
	Off-state in intermittent mode (Symbol is blinking)
	Intermittent mode
	Battery fully charged
	Info used time – shows runtime already used in days (📅) and hours (🕒) (Example for pump that has operated for 6 days and 23 hours). Not applicable for Invia Motion Endure.

## Change Invia Motion canister/tubing set

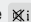Acoustic signals are muted by pressing the  button.

**1** Wash hands and prepare clean paper towel or tissue.

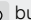**2** Remove the Invia Motion NPWT pump from the carrying case. Switch the Invia Motion NPWT pump off by pressing and holding the  button for 3 seconds. Press only once.

**3** Position the clamp next to the quick-connector and close the clamp on the pump tubing and also close the clamp on the dressing tubing.

**Clamp open**



**Clamp closed**



**4** Disconnect the Invia Motion NPWT pump tubing from the dressing tubing by pressing on the sides of the Quick-connector.



Use a paper towel or tissue to cover the tip of the used tubing to protect against wound fluid leakage.


**5** Detach the full canister/tubing set by rotating the canister to the left.



**6** Wash hands.

**7** Obtain and open a new Invia Motion canister/tubing set package.

### CAUTION

 Do not use an Invia Motion canister/tubing set if the sterile packaging is damaged or has expired.

 Make sure the correct canister is used: Ref 087.4016.

Contact your healthcare professional if the correct canister is not available.


**8** Remove the canister/tubing set from the packaging and attach it to Invia Motion NPWT pump.



**9** Connect the Invia Motion NPWT pump tubing to the dressing tubing. Push together until you hear a click.













**10** Open the clamp on the dressing tubing. Also make sure the clamp on the pump is open.

**11** Switch the Invia Motion NPWT pump on by pressing the  button. Slide the Invia Motion NPWT pump into the carrying case.

**12** Ask your clinician how to properly dispose of a used canister/tubing set.

## Notification

Symbol	Description	Troubleshooting
 blinking + acoustic signal	Leak in system	Check the system for leaks – refer to instructions for use, chapter Notifications and troubleshooting
 blinking + acoustic signal	Blockage	Check the system for blockage – refer to instructions for use, chapter Notifications and troubleshooting
 blinking + acoustic signal	Canister full	Change the Invia Motion canister/tubing set – refer to description in chapter "Change Invia Motion canister/tubing set"
 blinking + acoustic signal	Battery low	Connect the Invia Motion pump to the power supply and fully charge battery. Charging time can be up to 4 hours – refer to the instructions for use, chapter Notifications and troubleshooting
 blinking + acoustic signal	Battery fully discharged	Connect the Invia Motion pump to the power supply – refer to the instructions for use, chapter Notifications and troubleshooting
 blinking + acoustic signal	Battery missing	Ensure that battery compartment cover is mounted and closed properly – refer to the instructions for use, chapter Notifications and troubleshooting / Overview notifications
 blinking (example for 15 days version)	Pump time expired	Dispose of the Invia Motion NPWT pump. Attach another device if necessary – refer to the instructions for use, chapter Notifications and troubleshooting
 acoustic signal	Error (Internal fault)	Switch the Invia Motion NPWT pump off by pressing the  button and holding for 3 seconds. Patients: contact your healthcare professional immediately for further instructions. Refer to the instructions for use, chapter Notifications and troubleshooting / Overview notifications

Acoustic Signal can be muted by pressing 

**CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.**

 Medela, AG  
Lättichstrasse 4b  
6340 Baar | Switzerland

**USA**  
Medela LLC  
1101 Corporate Drive  
McHenry, IL 60050  
USA  
Phone +1 877 735 1626  
Fax +1 815 307 8942  
info-healthcare@medela.com  
www.medelahealthcare.com

**United Kingdom**  
Medela UK Ltd.  
Huntsman Drive  
Northbank Industrial Park  
Irlam, Manchester M44 5EG  
United Kingdom  
Phone +44 161 776 0400  
Fax +44 161 776 0444  
info@medela.co.uk  
www.medela.co.uk

**Canada**  
Medela Canada Inc.  
4160 Sladeview Crescent, Unit #8  
Mississauga, Ontario, L5L 0A1  
Canada  
Phone +1 905 608 7272  
Fax +1 905 608 8720  
info@medela.ca  
www.medela.ca

**International Sales**  
Medela AG  
Lättichstrasse 4b  
6340 Baar  
Switzerland  
Phone +41 41 562 51 51  
Fax +41 41 562 51 00  
ism@medela.ch  
www.medela.com

Medela wordmark and logo, Invia, and Invia Motion are registered in the U.S. Patent and Trademark Office.  
©Medela AG/ref. master doc. 101043358 A  
MUSA / 1548462 B 0421 / 101033541